LATITUDE™ Communicator Patient Literature Supplement Model 6359 USB Cellular Adapter

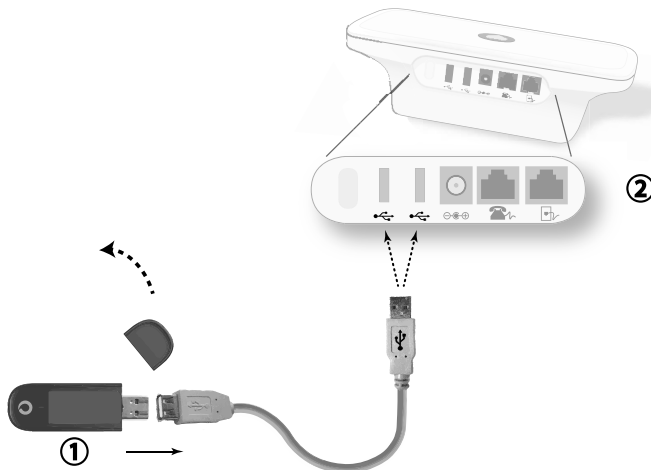
en

This patient literature supplement applies to Models 6280, 6290, and 6498 Communicators using the cellular data network option to send implanted device data to the LATITUDE Patient Management System.

Model 6359 USB Cellular Adapter is an alternative to the USB cellular adapter shown in your LATITUDE Communicator patient manual and quick start guide. (The patient manual and installation guide for the Model 6498 Communicator do not reference cellular adapters.) The following information applies to Model 6359:

- The “Where to Place Your USB Cellular Adapter” section in your patient manual also applies to Model 6359.
- It is normal for the LED light on the Model 6359 to blink. At times the light may be lit solid.
- A SIM card specific for Communicator use only has been pre-installed in Model 6359. Do not remove or tamper with this SIM card.
- Specifications listed in your patient manual for the USB cellular adapter do not apply to Model 6359. Refer to the specifications provided with Model 6359.

To set up the Model 6359 USB Cellular Adapter with your Communicator, follow the instructions provided with Model 6359 and refer to the figure below.

(1) Model 6359 USB Cellular Adapter; (2) Back of Communicator

For additional assistance, contact LATITUDE Patient Services at the phone number listed in your LATITUDE Communicator patient manual.

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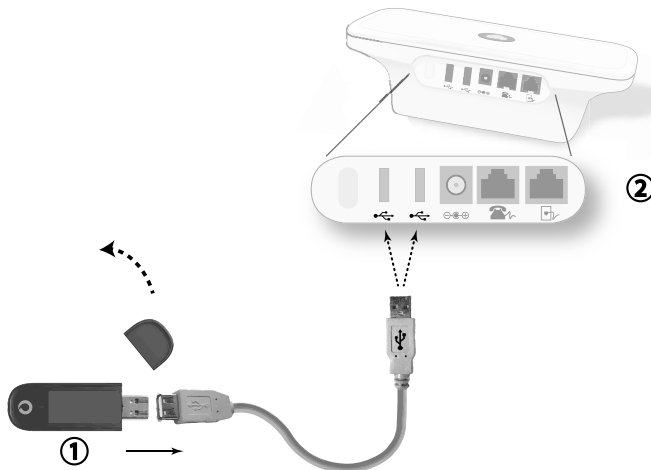
Patients and Families / Pacientes y familiares: 1.866.484.3268

www.bostonscientific.com

LATITUDE is a trademark of Boston Scientific Corporation or its affiliates. /

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© 2017 Boston Scientific Corporation or its affiliates. All rights reserved. /

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LATITUDE es una marca registrada de Boston Scientific Corporation o sus filiales.
© 2017 Boston Scientific Corporation o sus filiales. Reservados todos los derechos.